


MODEL 901S SERIES TWIN SADDLE STOOL ASSEMBLY INSTRUCTIONS

Parts Listing

- Seat Top
- Mechanism
- Gas Cylinder
- Base
- Base Cover
- (5) Casters or Non-Rolling Glides
- (2) M8 Washers (optional)
- (4) M8x16mm Screws
- Adjustable Foot Ring (optional)

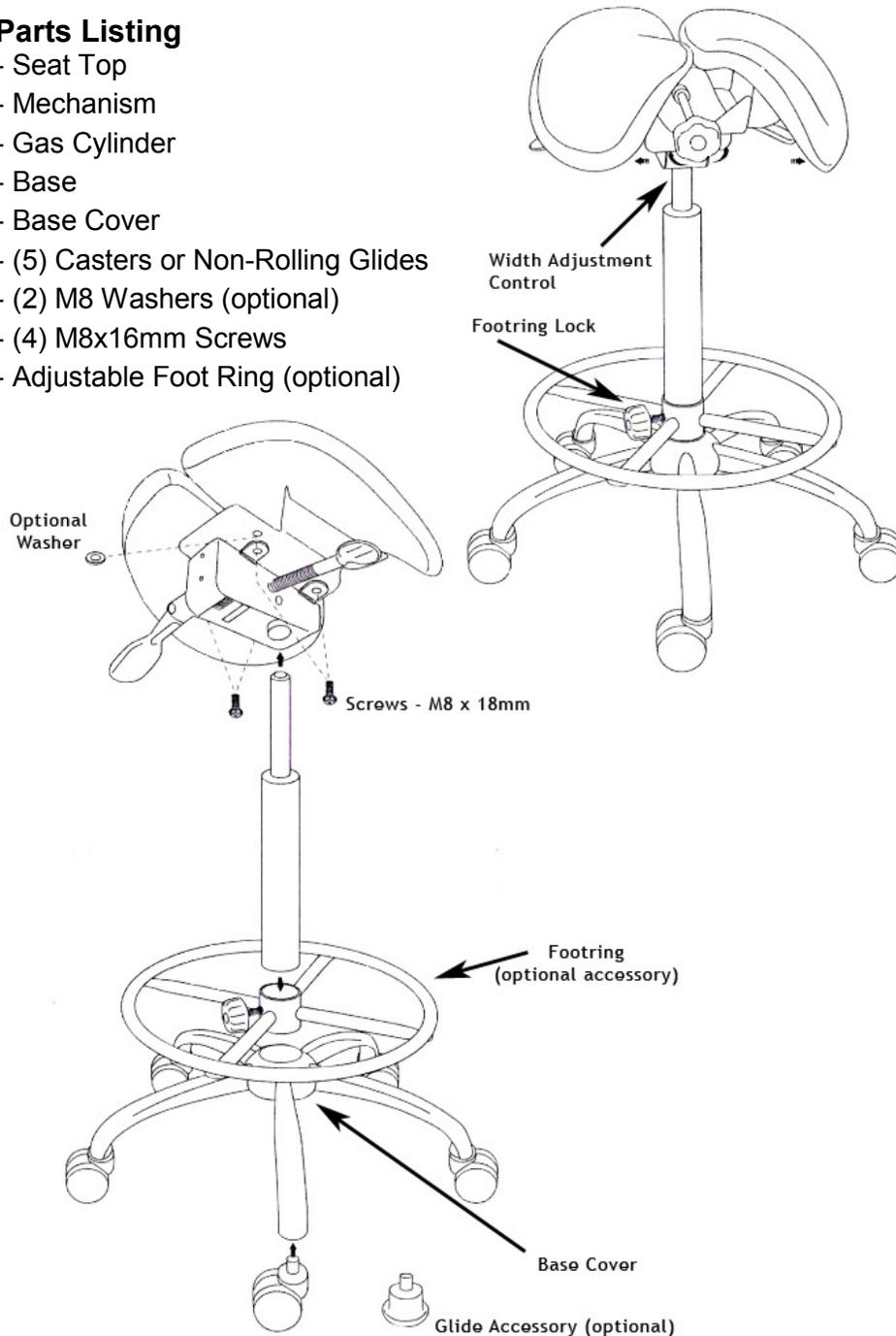

Please Read Instructions Before Assembly

** Tool required: Phillips head screwdriver or 12mm socket or wrench

Step 1: Insert each of the 5 Casters or Glides into the outer holes on the Base.

Step 2: Install the Base Cover over the center of the base by snapping the two halves together.

Step 3: (Models with Footring) Slide the Footring over the bottom of the Gas Cylinder, then insert the bottom end of the Cylinder into the center hole on the Base.

Step 4: Using 4 M8x16mm screws, connect the Mechanism to the Seat Top. Tighten screws securely.

Step 5: Remove the protective cover from the top of the gas cylinder and then place the Seat Top and Mechanism on top of the Gas Cylinder.
The mechanism should be securely seated over the cylinder top.

Note: If you prefer your seat angle to tilt further forward, the two rear M8x18mm screws can be removed and the optional M8 washers can be inserted between the Seat Top and the Mechanism before replacing and retightening the screws.

Adjustments:

Seat Height: Lift the Right lever handle to release the gas cylinder lock and raise or lower the seat to the desired height. Release the lever to re-engage the gas cylinder lock.

Seat Tilt: Lift the Left lever handle to release the tilt lock. Tilt seat to desired angle and return the lever to the DOWN position to re-engage the tilt lock.


Seat Width: Loosen the front Hand-wheel screw *counterclockwise* until the seat halves can move freely. Loosen only enough to release the seat halves. Do not over-loosen. Adjust the Width and Splay to the desired setting and tighten the Hand-wheel screw *clockwise*. Do not over-tighten

Caution: Do not use this chair as a step ladder.

Check for loose screws and tighten them every 6 months

Parts Listing For Back Rest Attachment

- Back Rest
- 2 x M8 x 16mm screws
- Washers (optional)

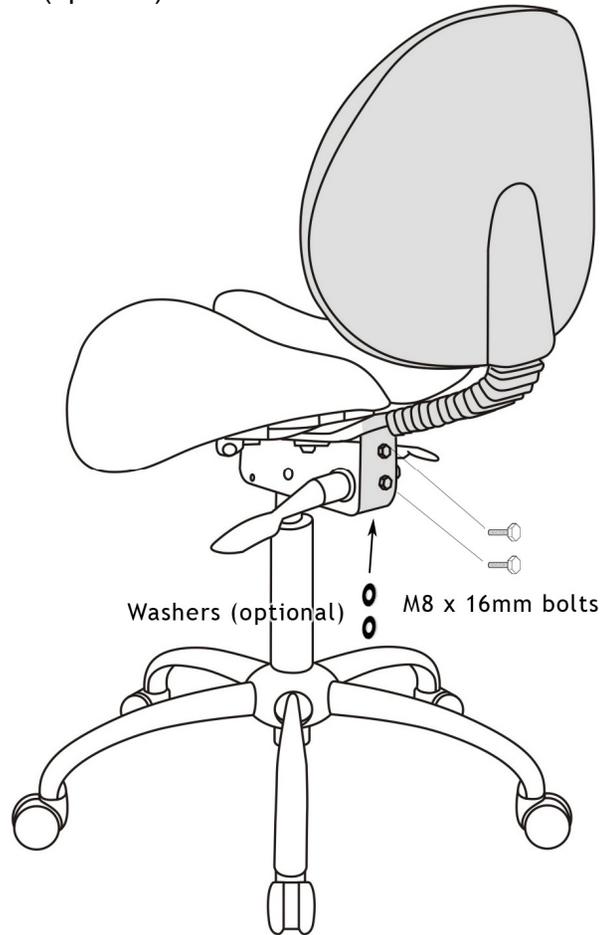

Please Read Instructions Before Assembly

** Tool Required: Phillips head screwdriver or 12mm socket or wrench

Prior to installation of Back Rest please complete Steps 1 through 5 as explained on the reverse of this assembly guide.

Back Rest Attachment:

Step 6: (*Models with Back*) - Attach the J-bar of the back rest to the two (2) rear holes on the seat mechanism with two (2) M8 x 16mm bolts. If desired, the back rest can be positioned further back by inserting one (1) or two (2) washers as spacers between the mechanism and the J-bar. Securely tighten all bolts.

Distributed in the United States by:

Health By Design
1260 Nimitz Ave, #670
Vallejo, CA 94592
888-909-3746 or 415-883-4550
www.healthysdesign.com

Distributed in Canada by:

Micwil Group of Companies
102 Wheeler Street
Saskatoon SK S7P 5A9
306-382-5995
www.micwil.com